

# Pregnancy Test Instructions


This test detects the presence of the hormone hCG, which is secreted during pregnancy. You may take the test as early as 2 days before a missed menstrual period. Test is most accurate with first morning urine specimens.

#1




Urinate (pee) into a cup

#2

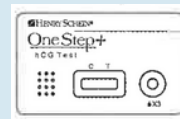

Remove the test device and pipette from the pouch. Place the test device on a flat surface.

#3

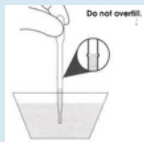

Squeeze the bulb of the pipette and insert the tip into the urine. Release the bulb and draw in urine to the fill line on the pipette. Do not overfill.

#4



Expel 3 drops of urine from the pipette into the sample well on the test device.

#5

Read results at 3-4 minutes.

## RESULTS INTERPRETATION:



**POSITIVE:** Two lines appear: 1 above "T": Test and 1 above "C":Control. Faint lines still indicate a positive test.



**NEGATIVE:** No line appears over "T" and 1 line is visible above "C".



**INVALID:** If no line appears over "C", the test is invalid.

Staff at UHS are available to support students in pregnancy counseling, including abortion referral and support.

To make an appointment at UHS, call 412-268-2157, option 2.